PRESSALIT®

R7634 - R7635
R7664 - R7665
R7834 - R7835
R7864 - R7865

PLUS

Shower seat 450 – Operation and maintenance manual	en
Brusesæde 450 – Brugs- og vedligeholdelsesvejledning	dk
Duschsitz 450 – Gebrauchs- und Pflegeanleitung	de
Siège de douche 450 – Manuel d'utilisation et d'entretien	fr
Douchezitting 450 – Gebruiks- en onderhoudshandleiding	nl
Duschsits 450 – Bruks- och underhållsanvisning	sv
Dusjsete 450 – Bruks- og vedlikeholdsveiledning	no
Asiento de ducha 450 - Instrucciones de uso y mantenimiento	es
Sedile per doccia 450 - Manuale d'uso e manutenzione	it
Siedzisko prysznicowe 450 – Instrukcja obsługi i konserwacji	pl
浴椅 450 - 操作和维护手册	中文

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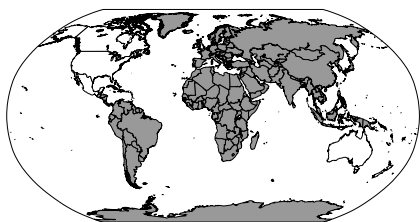
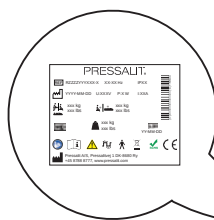
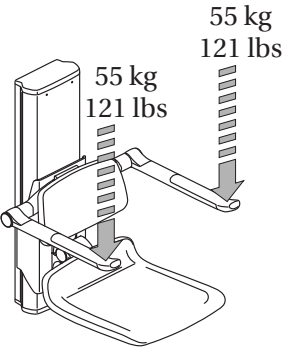
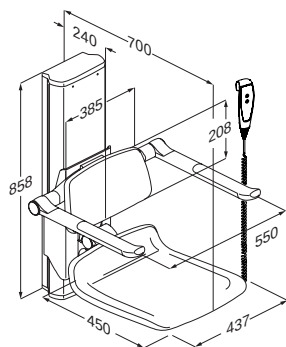
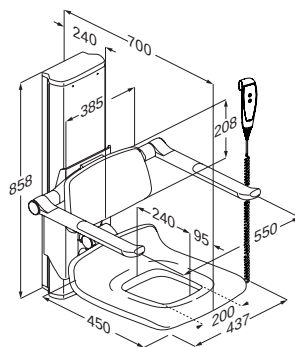
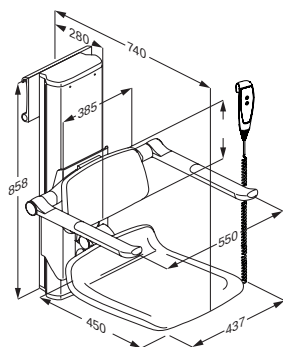
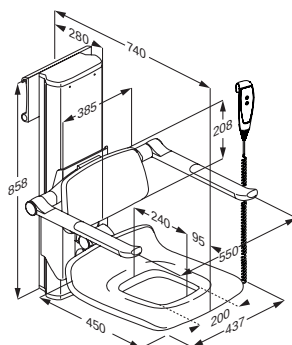
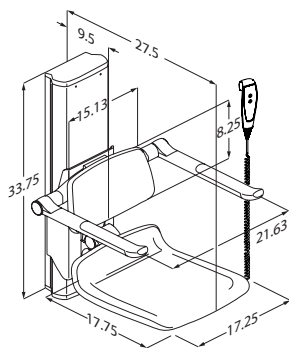
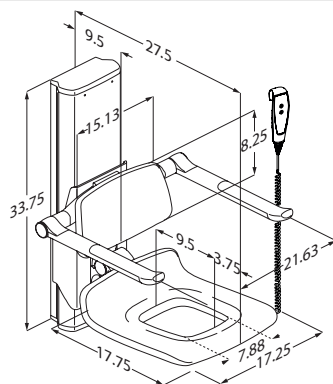
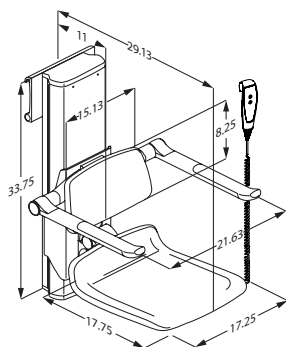
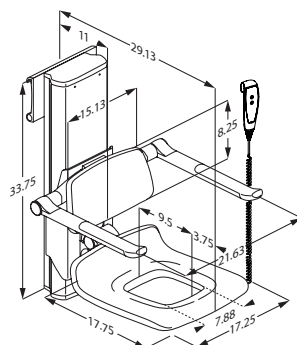
R7665, R7865

Content

Mounting instruction	7
Operational signage	7
Symbols used in this manual	8
General safety	9
Intended use	10
Intended user profile	10
Operating environment	10
Operating instructions	11
Folding the seats and armrests	11
Adjusting the shower seat	12
Adjusting the armrests	13
Adjusting sideways (R766X, R786X)	13
Cleaning	14
To remove limescale	14
Maintenance	15
Service inspection	15
Technical data	17
Product label	18
Trouble-shooting	20
Lubrication	20
Disposal and recycling	21

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

An operational signage can be downloaded from pressalitcare.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury
warning

Electricity or potentially
dangerous voltage

Head injury
warning

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

The shower seat is used to sit on during showering.

The armrests are intended for support when sitting and standing up from the shower seat.

Intended user profile

The shower seat can accommodate persons whose weight does not exceed

150 kg / 330 lbs.

The maximum vertical load for the armrests is 55 kg / 120 lbs.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals.

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

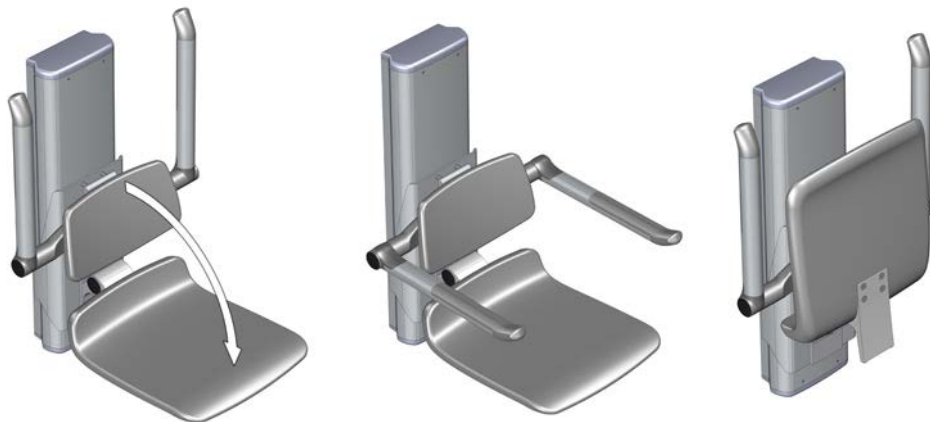
NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year.

Salt and chloride have a corrosive effect on the product's components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.

en

Operating instructions

Folding the seats and armrests

Fold the shower seat and the armrests fully down.

Fold it back up when not in use. You can hear a click when the shower seat and the armrests are correctly folded up.

en

Take care to avoid injury to hands and head when in the vicinity of closing moveable parts.

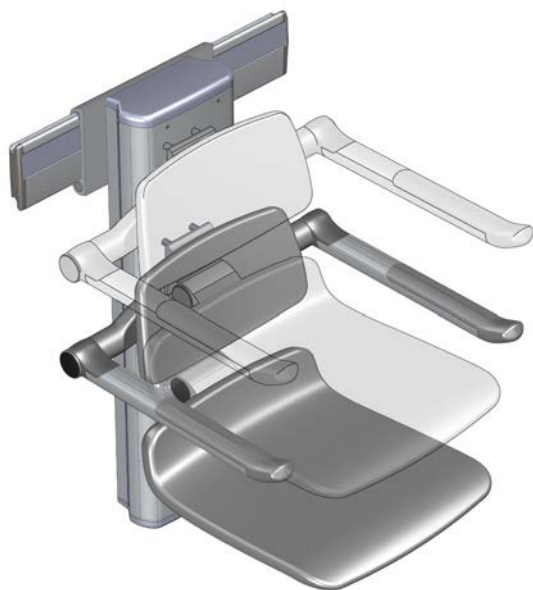
The armrests are not intended for horizontal load, e.g. as an aid for lateral transfer from a wheelchair to a shower seat.

IMPORTANT

Make sure that seat is dried off whenever it has been exposed to water.

Adjusting the shower seat

The shower seat can be adjusted 500 mm up and down.

Seat, backrest and armrests will adjust together

en

Take care to avoid injury to hands and head when in the vicinity of closing moveable parts.

The height adjustment is controlled by the wired hand control unit.

Press and hold the Up button to raise the shower seat, and press and hold the Down button to lower the shower seat. When the button is released, the motion stops.

CAUTION

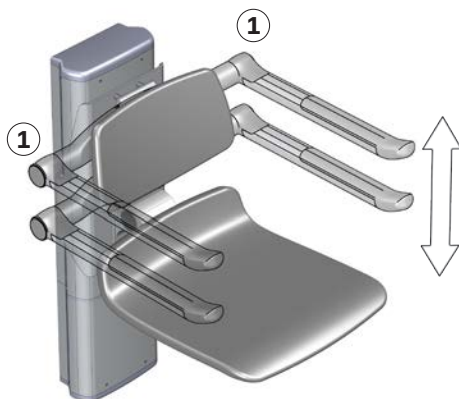
Never place objects underneath the shower seat.

Adjusting the armrests


Armrests can also be adjusted independently from seat and backrest.

To adjust the armrests upwards, pull upwards at the shoulders ① until desired height is reached and a click is heard.

To adjust the armrests downwards, pull the armrests all the way to the top, then support them all the way to the lowest position, and then pull upwards again to the desired position until a click is heard.

Adjusting sideways (R766X, R786X)

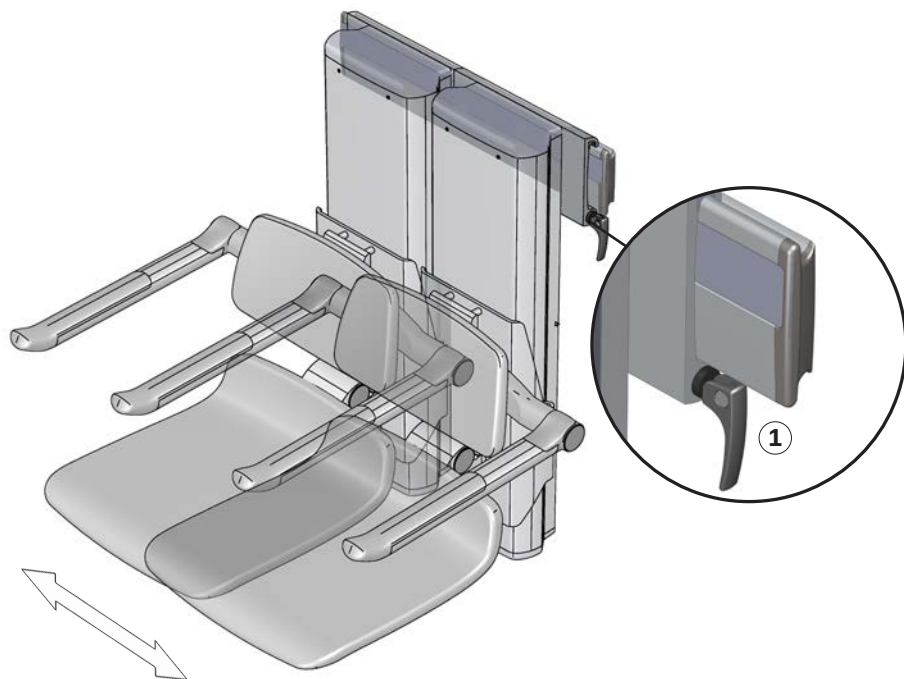Release the locking handle ① and move the shower seat sideways. Tighten the locking handle again to secure it in place.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the product is in use.

Maintenance

Expected service life of the shower seat is 10 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is also recommended to fold the seat and armrests up and down once a week.

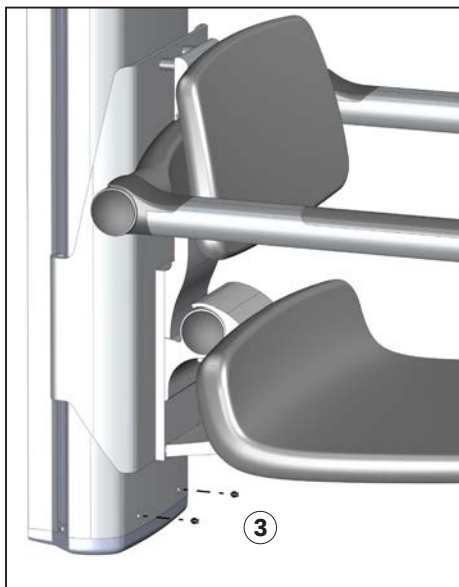
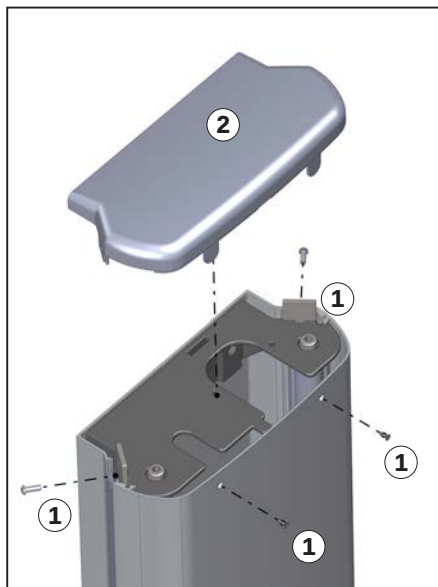
Service inspection

It is recommended that service inspection is done once a year.

Take the shower seat to its lowest position.


**Disconnect mains
before removing
the covers.**

Unscrew the 4 top screws ① and remove the top cover ②.

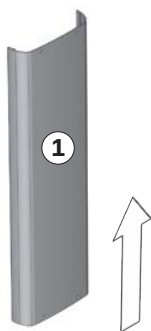Unscrew the 2 lower screws ③.

Pull the front cover ① upwards to remove it.

Remove any dirt from the inner rods ② using a cloth.

Lubricate the rods ② and the chain ③.

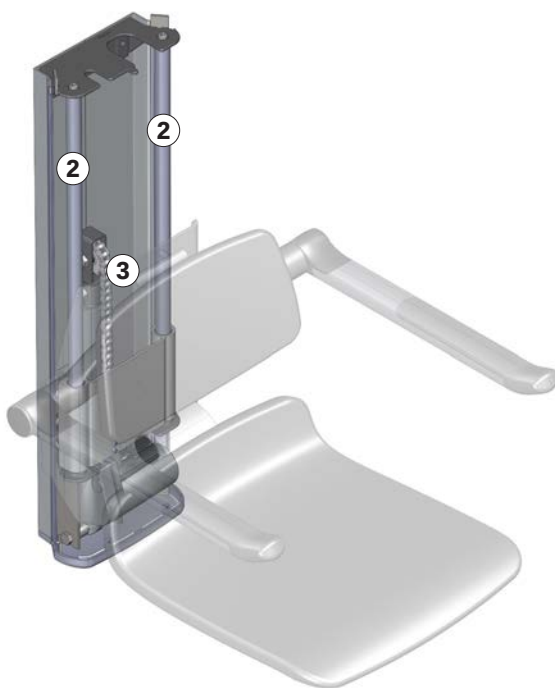
Ensure that all screws and bolts are properly tightened.

Reattach the front cover and the top cover.

Reconnect mains.

Run the shower seat to its full height, and then lower it to its lowest position to distribute the lubricant.

Check the functionality of the product.

en

Technical data

Materials	Anodized aluminium Stainless steel Polyurethane foam ABS SEBS
IP class	IPX6

Product label

- 19

⑩ IP class

⑪ Electric Current

⑫ UDI carrier GS1-128 barcode

⑬ Product weight

⑭ Batch code

⑮ UK Conformity Assessed

These products are CE-marked in accordance with relevant requirements in:

- ⑯
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
 - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

A declaration of conformity can be found at pressalit.com

⑰ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

⑱ Product name (if any)

⑲ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑳ Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock.

㉑ Duty cycle

㉒ Consult the manual for important cautionary information such as warnings and precautions.

㉓ Consult manual for use



㉔ Name and address of manufacturer

㉕ Class II Product, double insulated

Trouble-shooting


Symptom	Cause	Action
The product will not work, even though the controls is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The inner rods are not properly lubricated.	Oil the rods as described under "Maintenance".
The armrests and/or the seat is difficult to fold up and down.	The joints for the seat and the armrests needs lubrication.	Lubricate the joints for the seat and the armrests as described below.

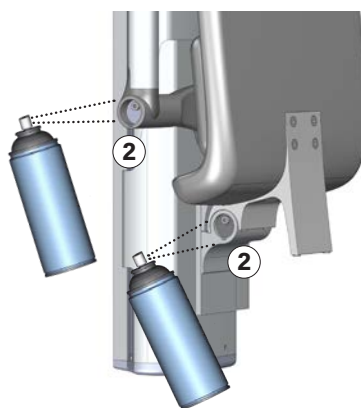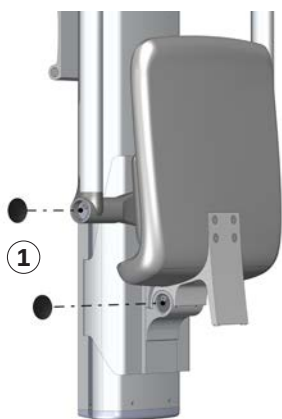Lubrication

Use a screwdriver to remove the cover ① for the joints.

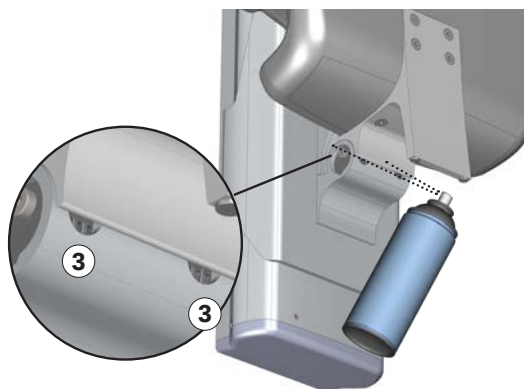Lubricate the joint ② and the two joint holes ③.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
SEBS	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy, which can be used for, for example, combined heat and power production.
ABS	As virgin ABS is somewhat expensive, recycling ABS is economically very attractive. Recycled ABS can be blended with virgin material to produce products with lower cost while preserving the high quality
Polyurethane	Like other plastics, polyurethane can be recycled in two primary ways: mechanical recycling, in which the material is reused in its polymer form, and chemical recycling that takes the material back to its various chemical constituents.

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